



# ANYLAB Rapid Diagnostic Test for pet

## Canine Coronavirus Antigen Test Kit (CCV Ag)

<b>Purpose</b>	Detection of canine coronavirus antigen
<b>Principle</b>	Immuno-chromatographic assay
<b>Specimen</b>	Canine feces and anus
<b>Sensitivity &amp; Specificity</b>	99.5% / 99.8%




Canine Coronavirus could infect although regardless of kind and age, especially puppies are more likely to be infected than adult dogs. The major symptoms of acute viral infectious diseases of the GI (gastrointestinal) tract include vomiting, diarrhea, dehydration, fatigue, etc. It is caused by contact with the feces and saliva of virus-infected dogs, or by stress. So it should be paid attention, because Canine Coronavirus is highly infectious among dogs who are fed in group, which could be spread rapidly.

### Features of AnyLab CCV Ag test


- Cost effective
- Easy to use – Mix & Drop then Read
- Fast results in 10 minutes
- 18 months shelf life at room temperature




### Test Procedure of AnyLab CCV Ag

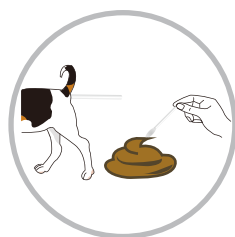


Collect feces and anus

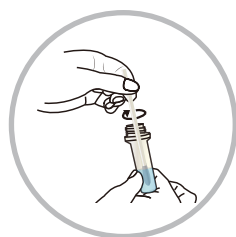

Insert the swab with specimen into the dilution buffer and stir swab at least 10 times

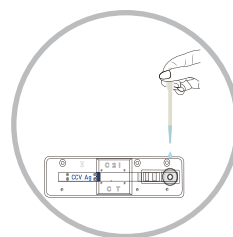

Add 4 drops of mixture into the sample well

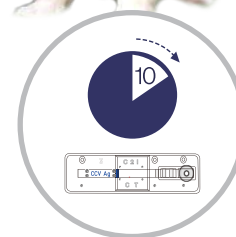

wait for 10 minutes at room temperature

order information

Number ZB201

item AnyLab CCV Ag Test

Testing time 10 minutes

Pack size 10 Tests / Box