

www.zetbio.com

ANYLAB Rapid Diagnostic Test for pet

Canine Distemper Virus Antigen Test Kit (CDV Ag)

Purpose	Detection of canine distemper virus antigen
Principle	Immunochromatographic assay
Specimen	Canine eye mucus
Sensitivity & Specificity	97.5% / 99.2%




Canine Distemper virus is also called measles virus, and spreads through saliva, feces, blood, sneezing and cough. It is a highly infectious virus, which has a high mortality rate. The infection could cause symptoms such as high fever, body fluids from the eyes and nose, vomiting and diarrhea, keratinization, of the nose and feet, etc. If secondary infection, it could lead to serious neurological symptoms, and the fatality rate is 80%. Symptoms such as degeneration of the nervous system due to sequelae after treatment, exercise-related functions are weakened mentally. Over time, the symptoms of severe visual loss, uncoordinated body movement, paralysis, and spasm are appeared.

Features of AnyLab CDV Ag test


- Cost effective
- Easy to use – Mix & Drop then Read
- Fast results in 10 minutes
- 18 months shelf life at room temperature




Test Procedure of AnyLab CDV Ag

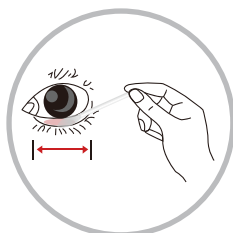


Collect Eye mucus

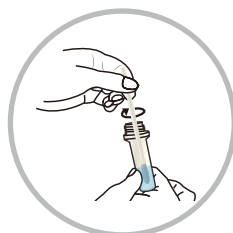

Insert the swab with specimen into the dilution buffer and stir swab at least 10 times

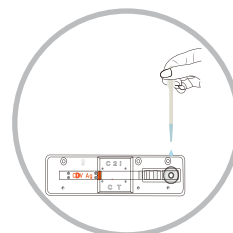

Add 4 drops of mixture into the sample well

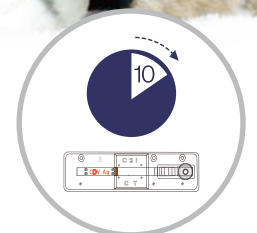

wait for 10 minutes at room temperature

Order Information

Number
ZB203

Item
AnyLab CDV Ag Test

Testing time
10 minutes

Pack size
10 Tests / Box