

www.zetbio.com

# ANYLAB Rapid Diagnostic Test for pet

## Feline Parvovirus Antigen Test Kit (FPV Ag)




<b>Purpose</b>	Detection of feline parvovirus antigen
<b>Principle</b>	Immunochromatographic assay
<b>Specimen</b>	Feline feces and anus
<b>Sensitivity &amp; Specificity</b>	96.5% / 99.0%

Feline Parvovirus is infected through mainly body fluids and feces of infected animals. Infected animals show signs of significant decrease in leukocyte. So It is also called Feline leukopenia. Typical Symptoms include hema feces, diarrhea, severe dehydration, malnutrition, anemia, etc, and other symptoms include fever and vomiting, depression, fatigue, appetite loss. Once an animal infected with this disease would die within 24 hours, due to its high mortality rate, treatment is needed immediately after disease diagnosis. Also, because of its high infectious, Isolation treatment is performed after hospitalization in ordinary animal hospital.

### Features of AnyLab FPV Ag test

- Cost effective
- Easy to use – Mix & Drop then Read
- Fast results in 10 minutes
- 18 months shelf life at room temperature


### Test Procedure of AnyLab FPV Ag

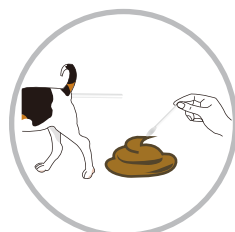


Collect feces and anus



Insert the swab with specimen into the dilution buffer and stir swab at least 10 times

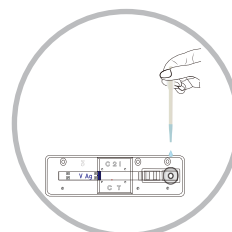

Add 4 drops of mixture into the sample well

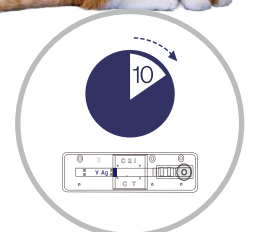

wait for 10 minutes at room temperature



order information

Number  
ZB301

Item  
AnyLab FPV Ag Test

Testing time  
10 minutes

Pack size  
10 Tests / Box