

## Preparation



Test Pouch (Test Device)



Disposable Dropper



Specimen Tube (Dilution Buffer)



Sterilized swab



Product Manual



Timer (Not Provided)



Disposable Gloves (Not Provided)



Caution : ONLY ONE TIME


## Test Procedure



**Check the expiry date.**  
If expiry date has passed, use another device.



Put on the gloves.

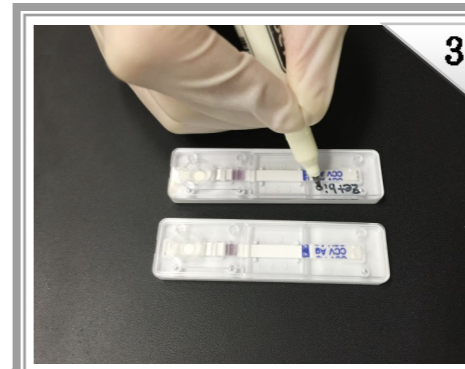

Open the test pouch and **write the patient's name** on the test device.




Collect the sample from anus or feces using the sterilized swab.




Insert the swab into specimen tube containing dilution buffer and mix by **10 times spin**.



Use the disposable dropper to collect mixed sample

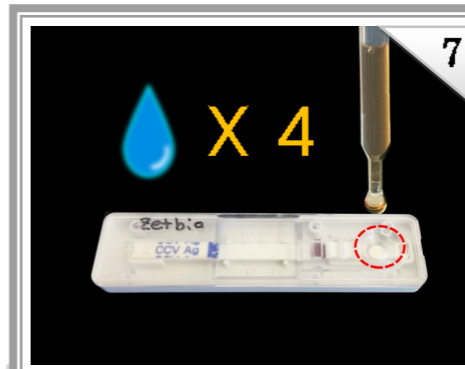


**Add 4 drops** of mixed sample into the sample well.

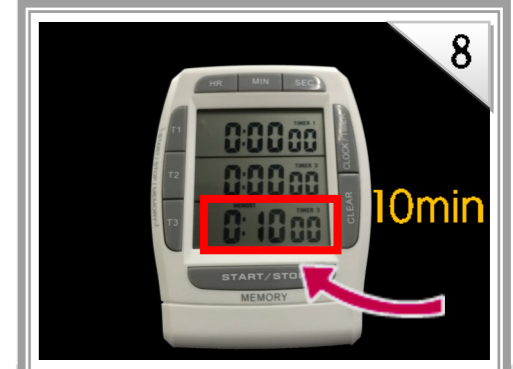

Read test result after 10 minutes **no later than 20 minutes**. : False positive can be shown after 20 minutes.

## Interpretation of Test result

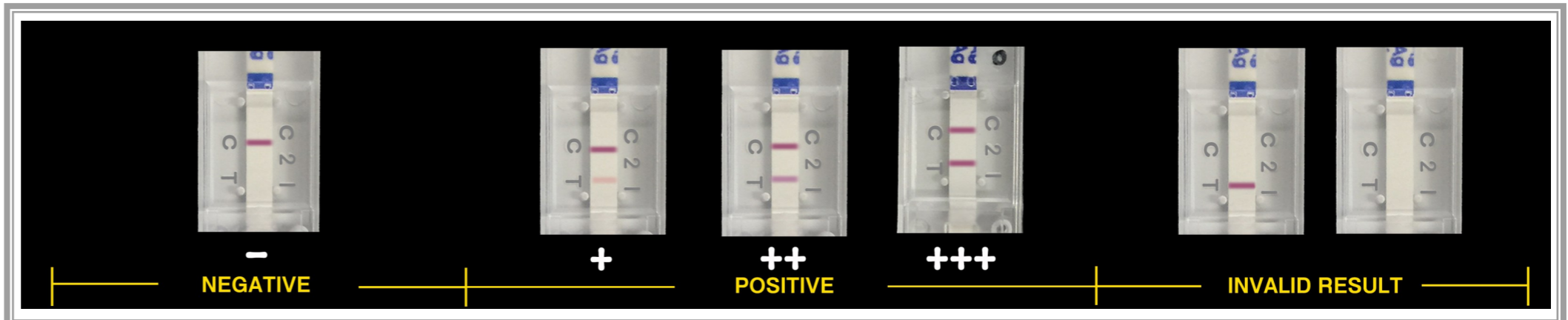

-Tel : +82-43-278-0075  
 -Fax : +82-43-278-0074  
 -E-Mail : support@zetbio.com  
 -office : #472, E9 Bldg., 1, Chungdae-ro, Seowon-gu, Cheongju-si, Chungcheongbuk-do, 28644, Rep.Korea  
 -Homepage : <http://www.zetbio.com>