

Preparation



Test Pouch (Test Device)



Disposable Dropper



Specimen Tube (Dilution Buffer)



Sterilized swab



Product Manual



Timer (Not Provided)




Disposable Gloves (Not Provided)



Caution : ONLY ONE TIME

Test Procedure

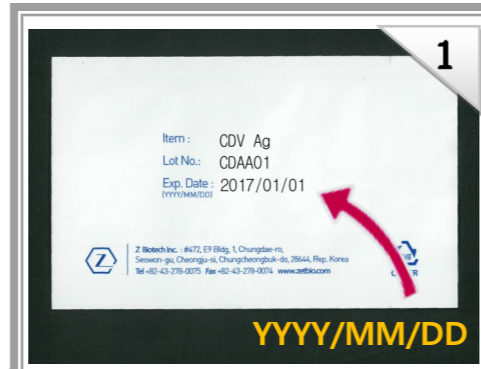


Check the expiry date.
If expiry date has passed, use another device.



Put on the gloves.



Open the test pouch and **write the patient's name** on the test device.

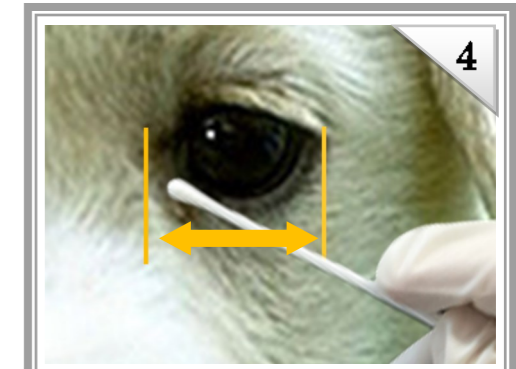


Collect the secretion of eye mucus using the sterilized swab.




Insert the swab into specimen tube containing dilution buffer and mix by **10 times spin.**



Use the disposable dropper to collect mixed sample

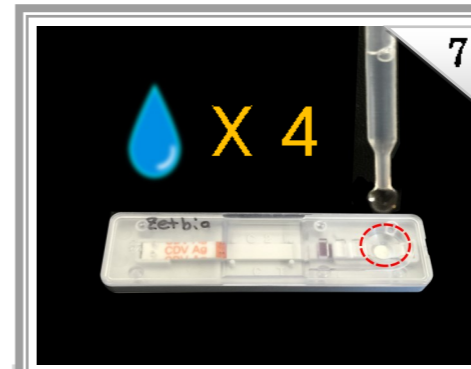

Add 4 drops of mixed sample into the sample well.

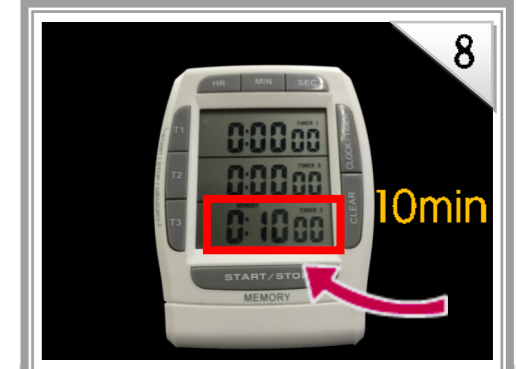

Read test result after 10 minutes **no later than 20 minutes.** : False positive can be shown after 20 minutes.

Interpretation of Test result



-Tel : +82-43-278-0075
-Fax : +82-43-278-0074
-E-Mail : support@zetbio.com
-office : #472, E9 Bldg., 1, Chungdae-ro, Seowon-gu, Cheongju-si, Chungcheongbuk-do, 28644, Rep.Korea
-Homepage : <http://www.zetbio.com>