

## Preparation



Test Pouch (Test Device)



Assay Buffer



Anticoagulant Tube



10ul Disposable Capillary Tube



Product Manual



Timer (Not Provided)




Disposable Gloves (Not Provided)



Caution : ONLY ONE TIME

## Test Procedure

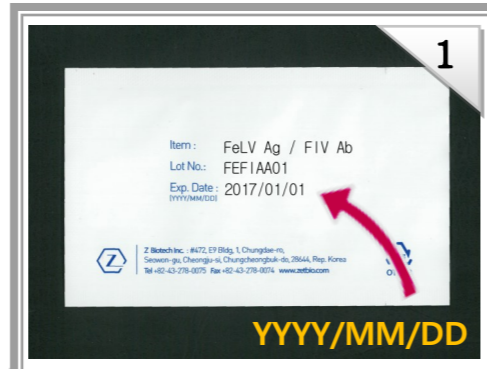


**Check the expiry date.**  
If expiry date has passed, use another device.



Put on the gloves.




Open the test pouch and **write the patient's name** on the test device.

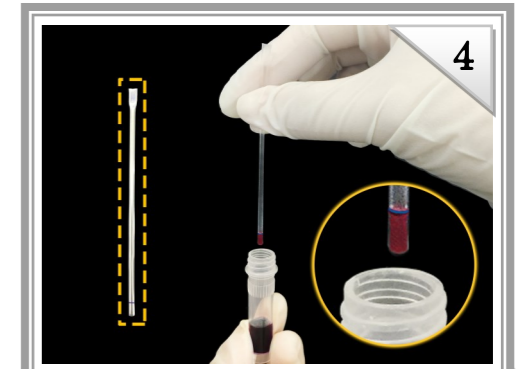

Use the disposable capillary tube to **collect sample 10ul**. Indicator line is 10ul




Add 10 ul of sample into the **sample well**.

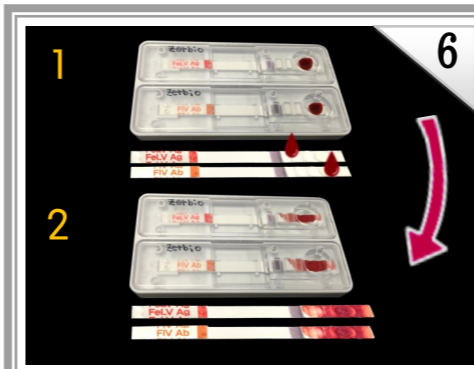

**Wait for 10~20 seconds.**  
It needs to absorb blood sample.




**Add 3 drops** of assay buffer into the sample well.

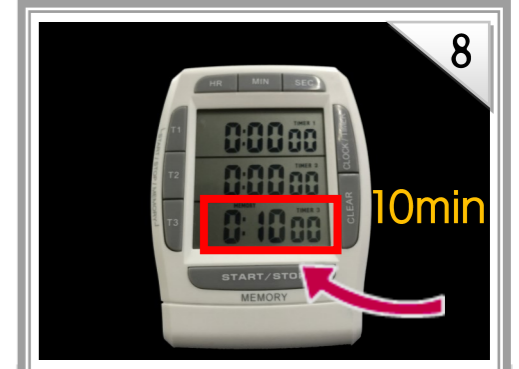

Read test result after 10 minutes **no later than 20 minutes**. : False positive can be shown after 20 minutes.

## Interpretation of Test result

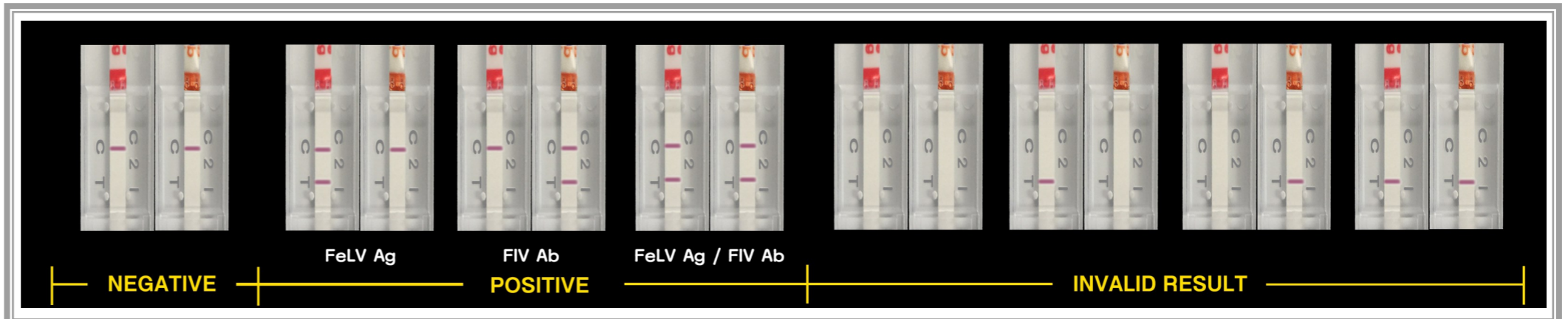

-Tel : +82-43-278-0075  
-Fax : +82-43-278-0074  
-E-Mail : support@zetbio.com  
-office : #472, E9 Bldg., 1, Chungdae-ro, Seowon-gu, Cheongju-si, Chungcheongbuk-do, 28644, Rep.Korea  
-Homepage : <http://www.zetbio.com>