

Preparation



Test Pouch (Test Device)



Assay Buffer



Anticoagulant Tube



10ul Disposable Capillary Tube



Product Manual



Timer (Not Provided)



Disposable Gloves (Not Provided)



Caution : ONLY ONE TIME

Test Procedure




Check the expiry date.
If expiry date has passed, use another device.




Put on the gloves.



Open the test pouch and **write the patient's name** on the test device.

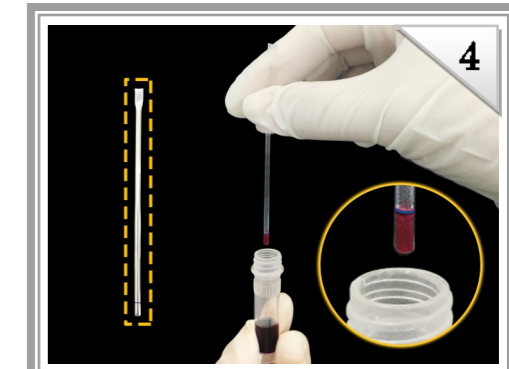


Use the disposable capillary tube to **collect sample 10ul**.
Indicator line is 10ul

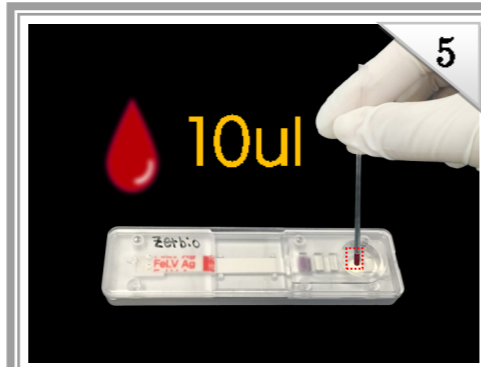

Add 10 ul of sample into the **sample well**.



Wait for 10~20 seconds.
It needs to absorb blood sample.

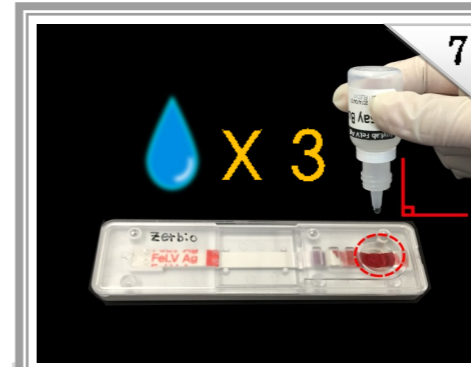

Add 3 drops of assay buffer into the assay buffer well.



Read test result after 10 minutes **no later than 20 minutes**. : False positive can be shown after 20 minutes.

Interpretation of Test result



NEGATIVE

+

POSITIVE

+++

INVALID RESULT

-Tel : +82-43-278-0075
-Fax : +82-43-278-0074
-E-Mail : support@zetbio.com
-office : #472, E9 Bldg., 1, Chungdae-ro, Seowon-gu, Cheongju-si, Chungcheongbuk-do, 28644, Rep.Korea
-Homepage : <http://www.zetbio.com>